KELO-COTE® SPRAY

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Brevet US n° 5 741 509

Brevet européen n° 1439866, 1409034

*Mustoe TA et al. Plast Reconstructive Surg 2002; 110:560-71

KELO-COTE® SPRAY

Scar therapy

Kelo-cote® Spray is a lightweight, self-drying silicone gel spray for the treatment of scars. Kelo-cote® Spray rapidly dries to form a sheet; this Silicone Gel Sheet layer is gas permeable, flexible and waterproof. Kelo-cote® Spray binds to the stratum corneum (the outer layer of dead skin cells) forming a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through normalized collagen synthesis cycles, and improves the physiological and cosmetic appearance of the scar.

Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars and keloids. The 2002 International Clinical Recommendations on Scar Management* recommended "silicone gel should be the first line of treatment in the initial management of scars and in the prevention of hypertrophic scars and keloids."

Kelo-cote® Spray can be used on joints, flexures, hairy areas and large area scars such as those caused from burns or major trauma.

Intended use:

Kelo-cote® Spray is intended for the management of old and new hypertrophic scars and keloids, resulting from general surgical procedures, trauma, wounds and burns.

Directions for use:

- Shake can well before use.
- Ensure that the affected area is clean and dry.
- Remove cap from the can
- Hold can 6 to 8 inches from surface to be treated.
- Spray along the length of the scar in a light, sweeping motion.
- Kelo-cote® Spray should be applied in a very thin, even coating with no holes or gaps on the affected area.
- Allow to air dry before covering with sun block or pressure garments.
- Kelo-cote® Spray should dry in 2-3 minutes. If the Kelo-cote® Spray is pooling, dripping or running you have sprayed too much. Gently remove the excess and allow the drying process to continue.

Kelo-cote® Spray should be applied twice daily for 90 days for maximum effect and can be applied under pressure garments and sunscreen. Recommended minimum treatment time is 60-90 days. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the initial 90 days.

Caution:

Kelo-cote® Spray contains flammable gases to help deliver the silicone.

- Extremely Flammable. Keep away from heat.
- Do not spray on open flame or incandescent material.
- Keep away from sources of ignition.
- Do not smoke while using.
- Pressurized container: Protect from sunlight and do not expose to temperatures exceeding 50°C/120°F.
- Do not pierce or burn can, even after use.
- Do not inhale Kelo-cote® Spray.

Do not use Kelo-cote® Spray near the eyes, nose or mouth. Avoid direct contact with mucous membranes, third degree burns and open wounds. Kelo-cote® may stain clothing if it is not completely dry. For external use only. If irritation occurs, discontinue use and consult your physician. Keep this product out of the reach of children. Store below 25°C/77°F. Do not use after the expiration (EXP) date printed on the can. The lot number is printed on the bottom of the can.

Ingredients: Butane, Pentane, Polysiloxanes, Silicon Dioxide.

Contents: 50ml & 100ml

www.kelocote.co.uk

Device Authorised Representative Alliance Pharma S.r.l. Viale Restelli 5, 20124 Milano, Italy

Manufacturers:

Advanced Bio-Technologies, Inc 4830 West Kennedy Blvd, Suite 600, Tampa, Florida 33609 USA

US Patent 5,741,509

Eur. Patent-Nr. 1439866, 1409034

*Mustoe TA et al. Plast Reconstructive Surg 2002; 110:560-71

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